

Article No.: 10-1541/01 Mobile Traction Stand
10-1542/01 to 10-1542/06 Raffia Finger Grip
10-1543/01 to 10-1543/03 Wire Finger Grip



Intended use / user group

The finger grips are intended for adjustment of the fingers or the hand in case of fractures. Simply by putting the grip on the finger and by hooking the grip in the traction bow with five chains you have the possibility to get the single members in the correct position by tractive force. The traction bow with the five chains will be fixed on a winch, which creates the tractive force.

Cleaning / Desinfection

The Raffia Finger Grips are disposable items. A sterilization or cleaning with desinfection solvent may destroy the material.

For the cleaning of the wire finger grips and the metal traction bow you can use commercially available desinfection solvents.

By steam sterilization or too hot cleaning a discolouring of the material may occur.

As soon as the grips show broken wires they may cause injuries. The grips have to be replaced.