



BiMobile[®] Dual Mobility System Cementless & Cemented

Product Rationale

System Description

Many years of experience with successful implant systems and fixation concepts as well as the latest material and coating technologies have been taken into account and used in the design of this dual mobility acetabular system. The result is the versatile Link[®] BiMobile[®] Acetabular Cup System.

The Link® BiMobile® Dual Mobility System exists in two versions:

A cementless and a cemented version. Both versions' metal shells are made from biocompatible and resilient EndoDur® CoCrMo material and are mirror polished on the inner surface to minimize wear.^{1,2}

BiMobile[®] Dual Mobility System – Shells, cementless

The cementless Link[®] BiMobile[®] Acetabular Cup is available with a TiCaP[®] double coating. The TiCaP[®] double coating combines a highly porous surface to achieve primary fixation and an osteoconductive³ calcium phosphate coating, which together ensure good primary and secondary implant stability.^{4,5}



BiMobile[®] Dual Mobility System – Shells, cemented

The cemented BiMobile® Acetabular Cup has a SatinLink® finished surface, which is well known from our SPII® stems. Latitudinal and longitudinal grooves strengthen the anchoring and allow air to escape during cementing.



BiMobile® Dual Mobility System - Liner

The Standard UHWMPE Liner can be combined with 22 mm or 28 mm CoCrMo or ceramic Link[®] Prosthesis heads.





TiCaP[®] Double Coating: Titanium (Ti)/Calcium Phosphate (CaP)

The TiCaP[®] coating is applied by first spraying a highly porous layer titanium of approximately 450 µm^{*} thick onto the surface of the implant using vacuum plasma technology. On top of this porous surface, an approx. 15 µm thick HX[®] layer of mechanically stable calcium phosphate is deposited in an electrochemical process. Animal experiments have demonstrated ingrowth on 47.9% of the implant surface.⁴

* The thickness of the titanium layers varies in different types of implant depending in their topographical application.





Rough Titanium-Plasma + HX[®] Coating TiCaP[®]

Double layer coating promotes bony ongrowth^{3,4}

Wide size range

- 42 mm 70 mm shell size
- 28 mm prosthesis heads starting from 48 mm shell size
- For a high ROM and a good anatomical reconstruction

Superior wear resistant shell

made out of biocompatibile and resilient EndoDur[®] CoCrMo material^{1,2,6}

Self-centering Liner

promotes even wear patterns and enhances dislocation resistance⁷

Strong retention

of prosthesis head in UHWMPE Liner⁸

Anatomic medio-ventral cut-out

- For reduced impingement and protection of psoas tendon and femoral nerve
- For high range of motion

Adoption of well known fixation concepts

- SatinLink® blasted surface
- Proven cement anchorage surface



Intraoperative Flexibility

with only one instrument Set for cemented and cementless version^{11,12}

Simple surgical experience

through streamlined, reliable and modern instruments with integrated color coding^{11,12}

Macro structure at equator ensures primary stabilit⁹

> **Mirror polished inner surface** for minimized wear and a prolonged implant life^{6,10}

Size-adjusted clearance

between metal shell and liner for consistent articulation¹¹

Optimized instrument design

The strong impactor-implant attachment and a good visualisation of implant rim ensures a safe and reproducible implantation.^{11,12} 2

The Dual Mobility Concept



The Dual Mobility concept was developed by Prof. Gilles Bousquet in the 1970s with the aim of avoiding recurrent hip luxations.¹³ The System is composed of a shell with a highly polished inner surface in which a mobile polyethylene liner having a pressed in prosthesis head is moving.

High Range of motion with less wear^{14,15,16}

The concept is based on two articulating surfaces where the prosthesis head articulates with the liner and the liner with the shell.

Primary Articulation

The small prosthesis head moves inside the liner until the neck of the stem impinges the liner

Secondary Articulation

The large polyethylen Liner moves freely inside the cup. This movement only occurs in extreme movements.

Reduced risk of dislocation^{15,17,18}

The combination of a small prosthesis head that is pressed into the large articulating liner incorporates a high jumping distance of Head/Liner unit in case of sub-luxation.





References (general)

- 1 Internal document W. LINK (DOC-08614)
- 2 Internal document W. LINK (DOC-08725)
- 3 Ullmark G et al.: "Analysis of bone formation on porous and calcium phosphate-coated acetabular cups: a randomised clinical [18F] fluoride PET study." Hip International 22.2 (2012).
- 4 Cunningham B W et al.: "General Principles of Total Disc Replacement Arthroplasty", Spine, Vol. 28, No. 20 Suppl., 2003
- 5 Bobyn, J. D., et al. "The optimum pore size for the fixation of porous-surfaced metal implants by the ingrowth of bone." Clinical orthopaedics and related research 150 (1980): 263-270.
- 6 Long, M., & Rack, H. (1998). Titanium alloys in total joint replacement—a materials science perspective. Biomaterials, 19(18), 1621-1639.
- 7 Fabry C, Kaehler M, Hermann S, Woernle C, Bader R. 2014. Dynamic behavior of tripolar hip endoprostheses under physiological conditions and their effect on stability. Medical Engineering & Physics 36:65–71.
- 8 Internal document W. LINK (DOC-08553)
- 9 Internal document W. LINK (DOC-08695)
- 10 Loving L, Herrera L, Banerjee S, Heffernan C, Nevelos J, Markel DC, Mont MA. 2015. Dual mobility bearings withstand loading from steeper cup-inclinations without substantial wear. J Orthop Res. 33(3):398-404.
- 11 Internal document W. LINK (DOC-08847)
- 12 Internal document W. LINK (DOC-07974)
- 13 Noyer, D., Canton, J. H. (2016). Once upon a time... Dual mobility: history. International Orthopaedics Vol. 41 No. 3 (March 2017): 611-618
- 14 Charnley, John. "The long-term results of low-friction arthroplasty of the hip performed as a primary intervention." Bone & Joint Journal 54.1 (1972): 61-76.
- 15 Philippot, R., Camilleri, J. P., Boyer, B., et al. (2009). The use of a dual-articulation acetabular cup system to prevent dislocation after primary total hip arthroplasty : analysis of 384 cases at a mean follow-up of 15 years. SICOT 33: 927-932.
- 16 Wroblewski, B., Siney, P., Flemin, P. (2009). The principle of low frictional torque in the Charnley total hip replacement. JBJS (Br) Vol. 91-B(7): 855-858.
- 17 Stroh, D. Alex, et al. "Dual-mobility bearings: a review of the literature." Expert review of medical devices 9.1 (2012): 23-31.
- 18 Nevelos, J., Bhimji, S., Macintyre, J., et al. (2010). Acetabular Bearing Design Has a Greater Influence on Jump Distance than Head Size. 56th Annual ORS Meeting: Poster #2028.

© LINK • 679_BiMobile_Product_en_2018-02_003

Waldemar Link GmbH & Co. KG

Barkhausenweg 10 · 22339 Hamburg · Germany Phone +49 40 53995-0 · info@linkhh.de www.linkorthopaedics.com

